




TOGETHER AT LAST

PERI-STRIPS DRY WITH SECURE GRIP
FOR THE DA VINCI X[®] AND XI[®] SURGICAL SYSTEMS

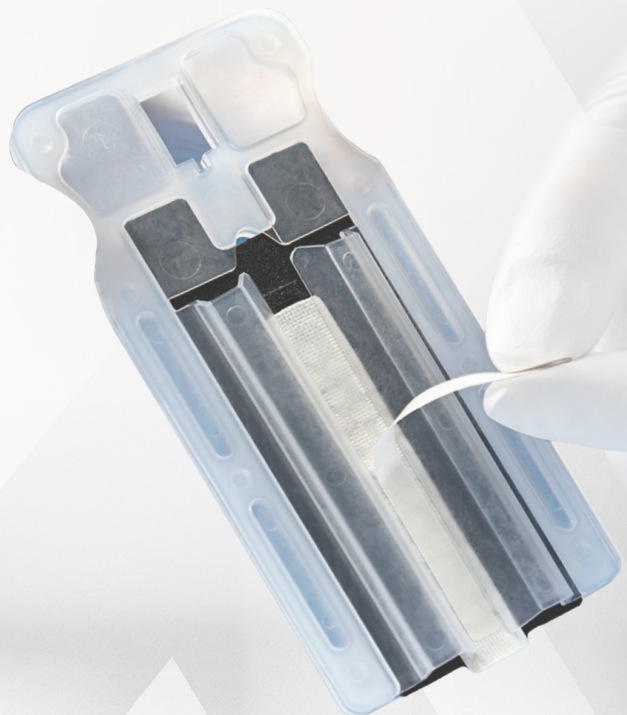

Baxter

**SIMPLY PEEL,
 SECURE, & PASS**



1. OPEN

In a sterile field, open inner pouch using aseptic technique and discard ship clip.




2. PEEL

Peel liner from both Anvil and Cartridge sides.



3. SECURE

Position open stapler jaws with corresponding sides of the loading unit and advance until fully seated. Close stapler and discard plastic sheath.

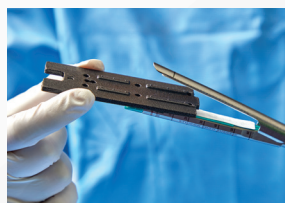

4. PASS

Open the stapler jaws to remove foam insert. Close stapler and pass to surgeon.

TOGETHER AT LAST

PERI-STRIPS DRY Staple Line Reinforcement with VERITAS Collagen Matrix now comes with SECURE GRIP Technology (PSDV SECURE GRIP), arming you with the power of a biologic material you can rely on combined with the convenience of a faster, simpler preparation.¹

In addition, PSDV SECURE GRIP is now available for use with the *da Vinci X®* and *XI®* Surgical Systems' crystal clear 3DHD vision, access and surgeon control and autonomy.^{2,3}

1/2
THE PREPARATION
TIME

FASTER PREPARATION TIME³

In a Staple Line Reinforcement Comparison Study, 30 bariatric nurses and surgical technicians evaluated **Stapler Preparation Time, Ease of Use** and **Learnability** of PSDV SECURE GRIP, PSDV Gel and SEAMGUARD.

PSDV SECURE GRIP resulted in a **54.55% reduction in preparation time** when compared to SEAMGUARD® and a **59.67% reduction in preparation time** when compared to PSDV Gel.



ORDERING INFORMATION

PRODUCT CODE	PRODUCT DESCRIPTION	COMPATIBLE STAPLERS
PSDA60GIA	6 units/box	Intuitive SureForm™ 60 Stapler for da Vinci X® and da Vinci Xi®
PSDA60GIATHN	6 units/box	

Indications and Important Risk Information for PERI-STRIPS DRY with VERITAS Collagen Matrix with SECURE GRIP Technology

PERI-STRIPS DRY with VERITAS Collagen Matrix with SECURE GRIP Technology Indications for Use

PERI-STRIPS DRY with VERITAS Collagen Matrix with SECURE GRIP Technology (PSDV SECURE GRIP) is intended for use as a prosthesis for the surgical repair of soft tissue deficiencies using surgical staplers when staple line reinforcement is needed.

PSDV SECURE GRIP can be used for reinforcement of staple lines during gastric, bariatric, small bowel, colon and colorectal procedures.

Contraindications

The use of PSDV SECURE GRIP is contraindicated in patients with known sensitivity to bovine or acrylic material.

Warnings and Cautions

Do not re-sterilize. Do not use PSDV SECURE GRIP if the product has been exposed to solutions above room temperature, to chemicals, antibiotics or other substances other than specifically addressed in the instructions, or if the heat indicator has been activated. The use of PSDV SECURE GRIP has not been studied in lung or bronchus resection.

For questions or ordering information, go to advancedsurgery.baxter.com or contact your local Baxter representative.

ADVANCING THE ART OF HEALING

RX Only. For safe and proper use of this device, please refer to the Instructions for Use.

References: 1. PERI-STRIPS DRY Staple Line Reinforcement with VERITAS Collagen Matrix with Secure Grip Technology Instructions for Use. 2. da Vinci Xi Surgical System. Available at: <https://www.intuitive.com/en-us/patients/da-vinci-robotic-surgery>. 3. Baxter data on file. Baxter, Peri-Strips Dry, Secure Grip and Veritas are all registered trademarks of Baxter International, or its subsidiaries. da Vinci Xi Surgical System is a registered trademark of Intuitive Surgical. GBU-AS28-230001 v1 April 2023